

Tuff Tread

502 W. Montgomery STE 120
Willis, TX 77378
PH: (800)827-2017 FAX: (888)898-8974
www.SuperTuff.com

Universal Lift Motor Potentiometer Replacement

The lift motor potentiometer is located on the topside of the lift motor.

TO REMOVE: Take careful note on which direction the wires are pointing. Using a Phillips screwdriver, remove the two screws from the bracket holding the potentiometer in place.

Now that you have the potentiometer off the lift motor, you need to remove the bracket so you can use it on the new one. To remove it, you simply take off the jam nut that holds the bracket on. (KEEP THE NUT AND THE BRACKET)

Compare the old potentiometer to the new Universal Potentiometer Replacement to determine if the stem is the same size. You may need to carefully grind or file the stem on the Universal Potentiometer Replacement to match the length of the old potentiometer stem.

TO INSTALL: Put the bracket on the new potentiometer and snug the jam nut down. Make sure your bracket is positioned in the same direction it was when it came off the old one.

Make sure your wires are pointing in the same direction as the ones on the old potentiometer. Line up the flat side on the potentiometer shaft and the flat side of the hole and push into place.

Line up the two screw holes and put in the screws

Recalibrate the display using the enclosed recalibration instructions.